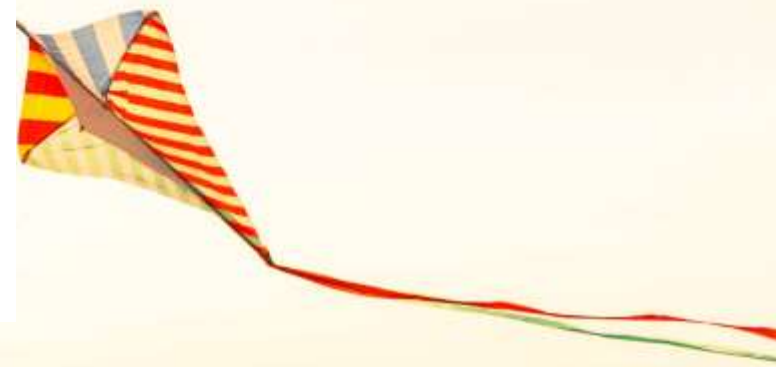
Distributor of *Medilime*
Tummy Button

PEG zaken
5 November 2019

Harm Meijer

Inhoud

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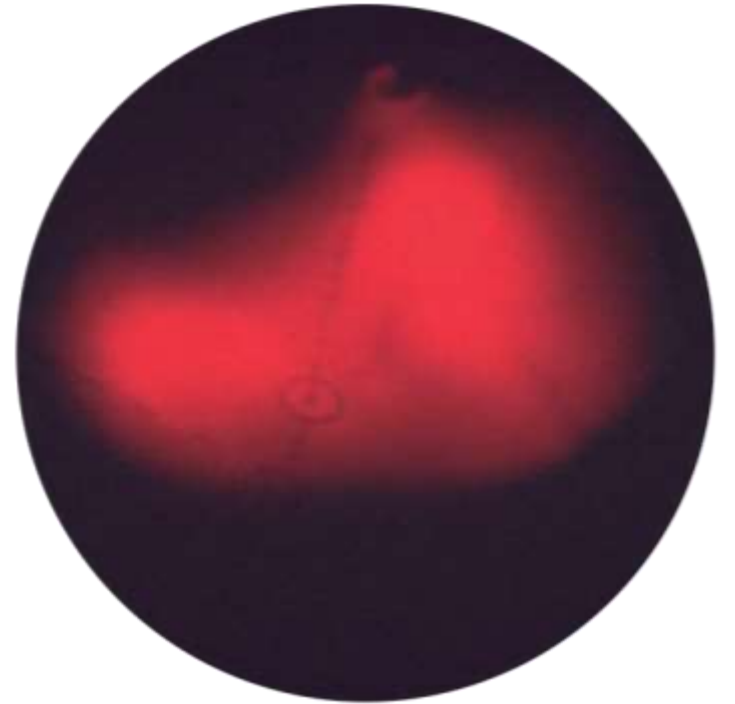
PEG



- **Percutaneous Endoscopic Gastrostomy**

- “The room was dark, and a glow emanated from the abdomen of a tiny baby on the operating table.” - *Dr. Jeffrey Ponsky*

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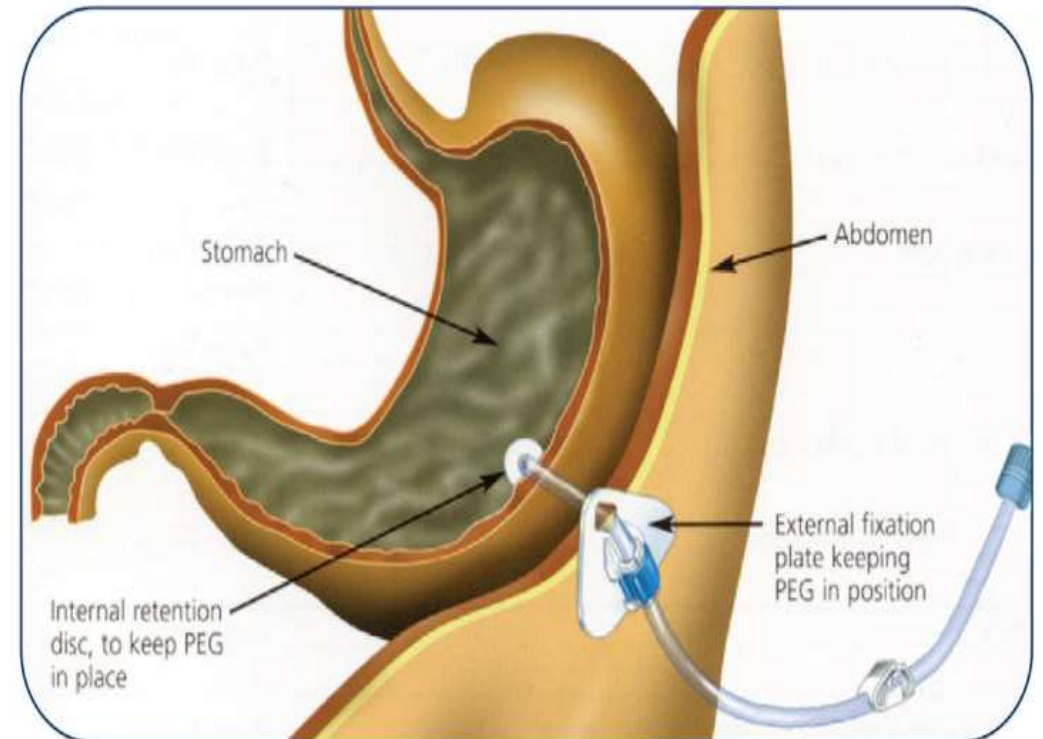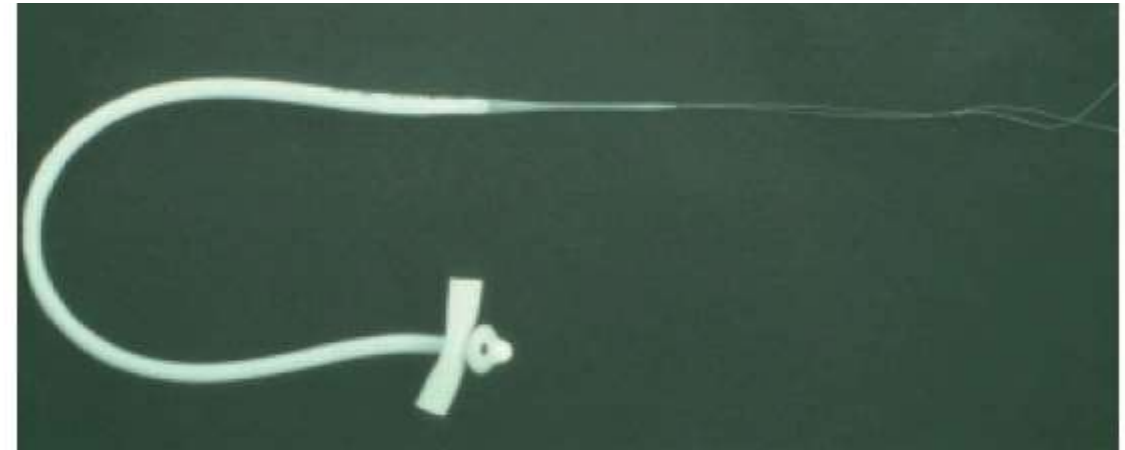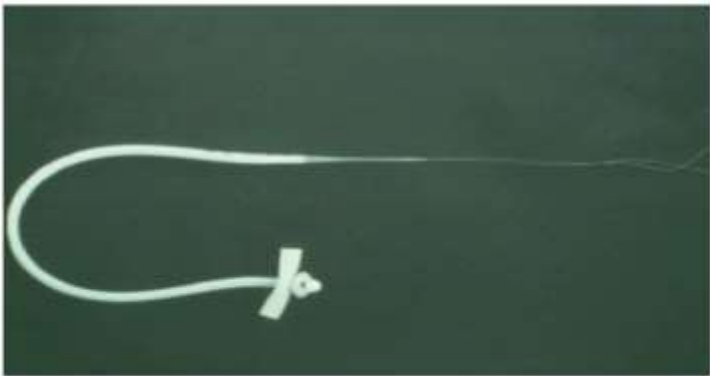
PEG's

Na de PEG ?

- When the PEG has to be changed
- Replacement tubes (Medilime gastric tube)
- Buttons (Medilime Tummy Button)

Design

Waarom ENFit ?

NY Times

MONEY & POLICY

U.S. Inaction Lets Look-Alike Tubes Kill Patients

By GARDINER HARRIS AUG. 20, 2010

Thirty-five weeks pregnant, Robin Rodgers was [vomiting](#) and losing weight, so her doctor hospitalized her and ordered that she be fed through a tube until the birth of her daughter.

But in a mistake that stemmed from years of lax federal oversight of medical devices, the hospital mixed up the tubes. Instead of snaking a tube through Ms. Rodgers's nose and into her stomach, the nurse instead coupled the liquid-food bag to a tube that entered a vein.

Putting such food directly into the bloodstream is like pouring concrete down a drain. Ms. Rodgers was soon in agony.

"When I walked into her hospital room, she said, 'Mom, I'm so scared,' " her mother, Glenda Rodgers, recalled. They soon learned that the baby had died.

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International
Organization for
Standardization

Enhancing Patient Safety

New global design standards
for enteral device tubing connectors

Systeem

The new design standard impacts the entire enteral feeding system

PATIENT-ACCESS END

SYRINGE (CURRENT)

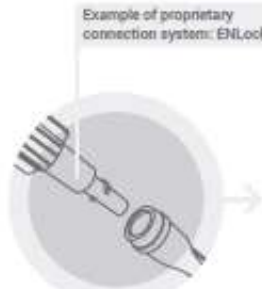
Syringe with ENFit connector

SYRINGE (FINAL)

Syringes to administer medicine, flush, hydrate, or bolus feed through enteral tubes will now require a precise enteral-specific fitment.

MEDICATION PORT

Administration sets with medication ports will have the male ENFit connector.

Example of proprietary connection system: ENLock

Stepped or Christmas tree/funnel connector

New ENFit female connector

ENFit Transition to ENLock

ENFit Transition Connector

New ENFit female connector

FEEDING TUBE (FINAL)

Changing from the stepped or proprietary connectors (e.g. ENLock) to the new ENFit female connector. The feeding tube port for the administration set will change to the new ENFit male connector.

FEEDING TUBE (CURRENT)

TRANSITION SET (TEMPORARY)

Suppliers will provide transition connectors, like the two pictured above, to allow fitment to current feeding port until new ENFit enteral feeding tubes are available. Estimated conversion and phase out of transition connector is one year.

NUTRITION END

ENPLUS OR
40mm SCREWCAP CONNECTOR
(Proposed ISO 18250)

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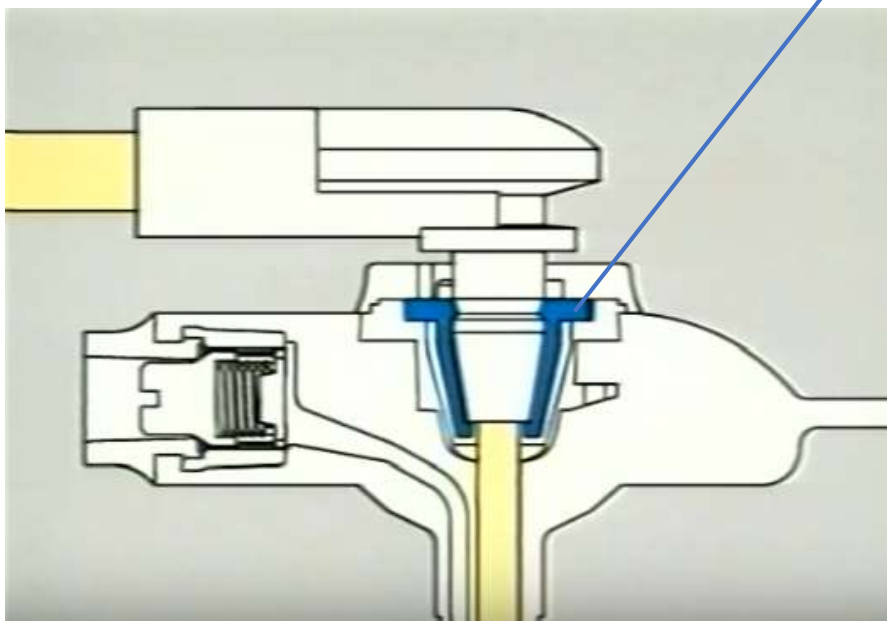
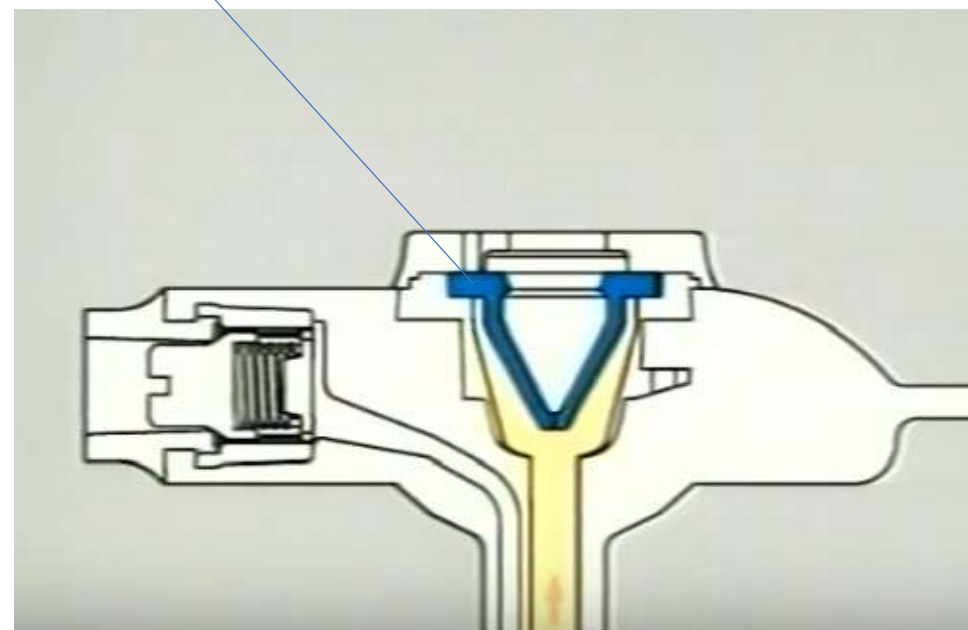
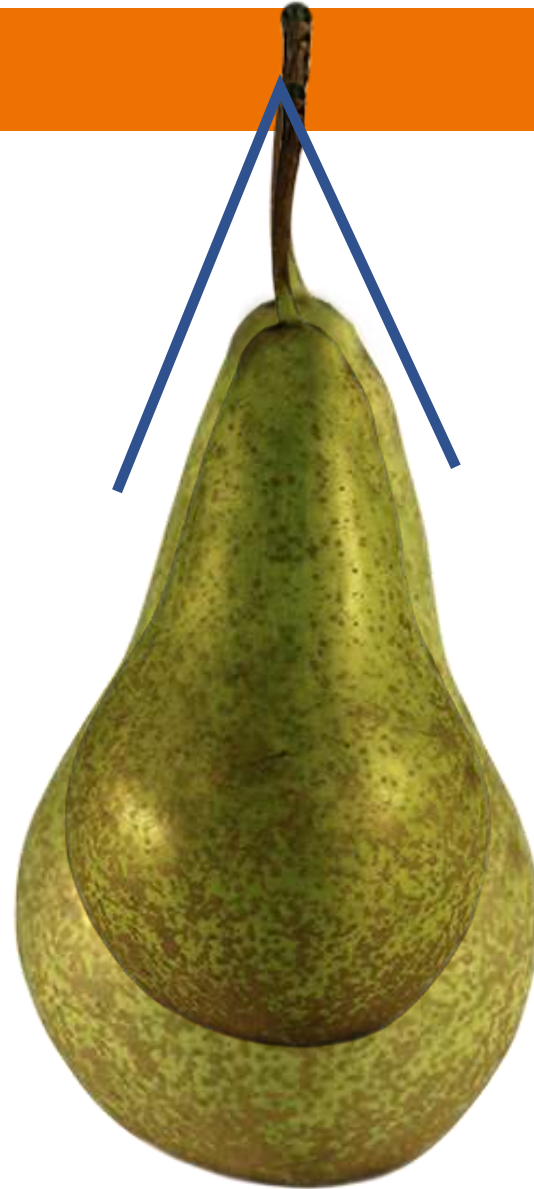
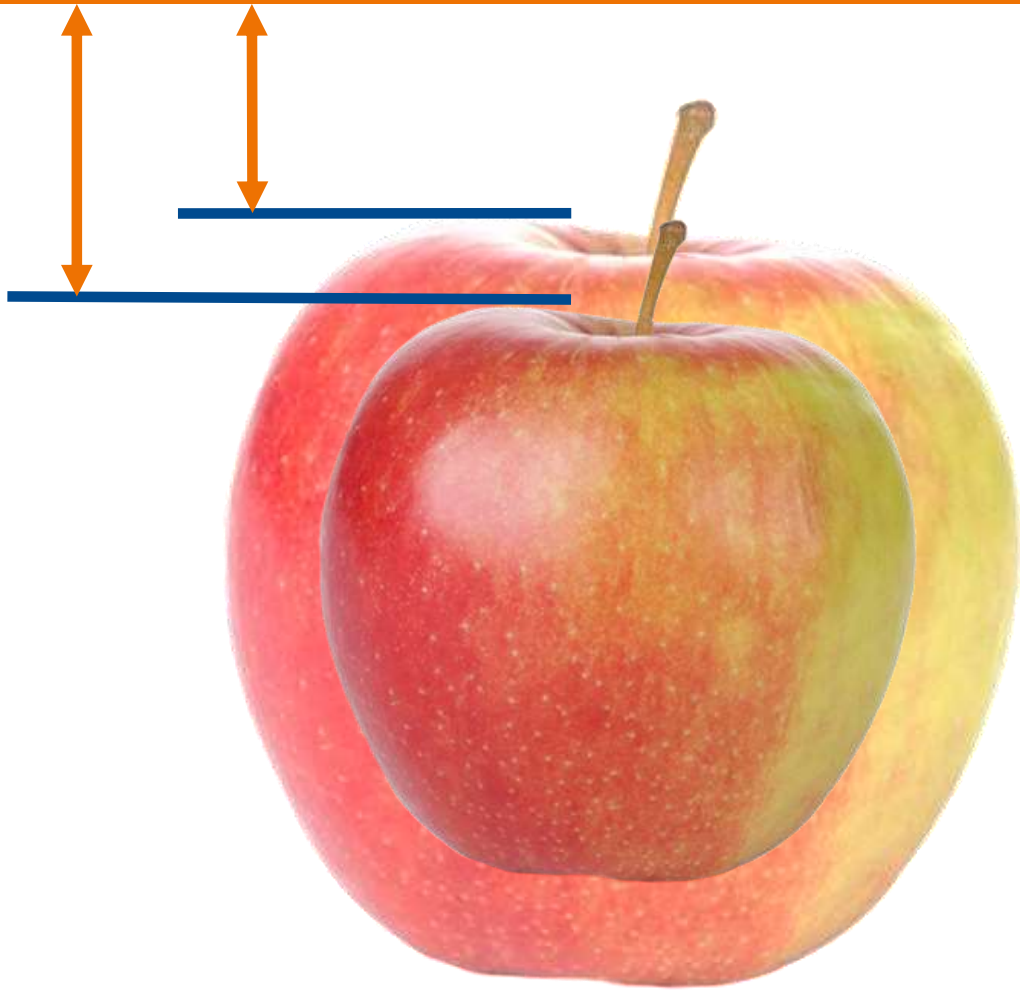
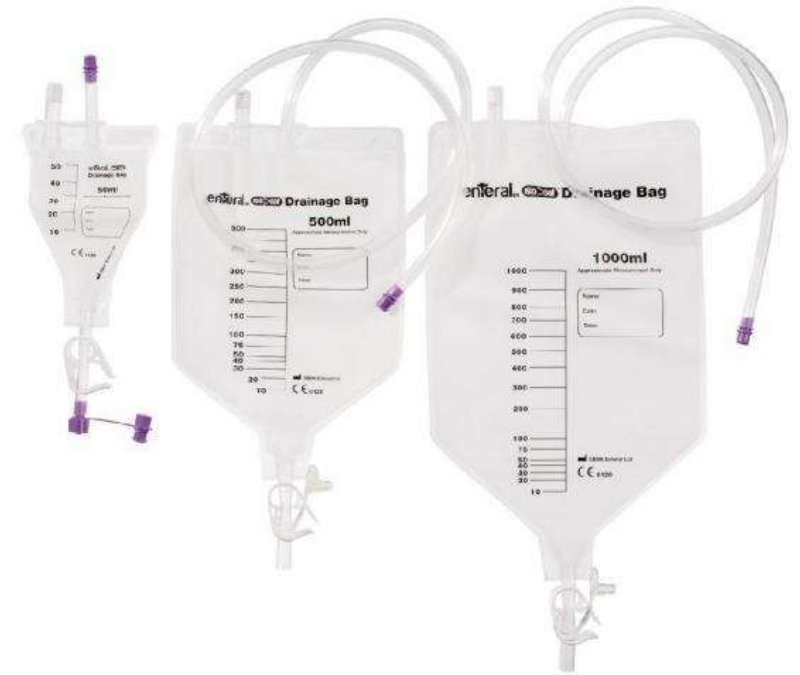
ENFit® Cleaning Procedures

Feeding Tubes with Male ENFit Connectors

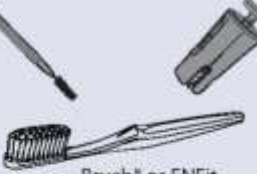
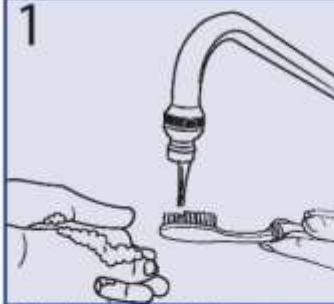
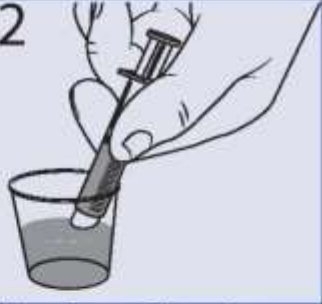
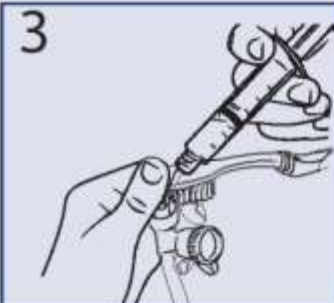
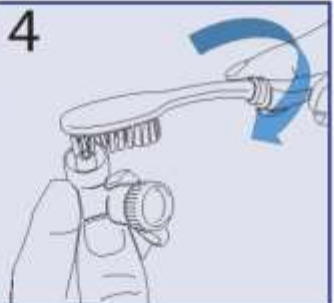
(e.g. Nasogastric, Transpyloric, Orogastric, Percutaneous Endoscopic Gastrostomy Tubes and other ENFit devices)

Tips for keeping ENFit feeding tube ports clean. Inspect before you connect!

- **Priming Feeding Sets** - Stop priming before fluid reaches the end of the tube.
- **ENFit Syringe Draw Up** - Wipe medication and nutrition from tip/outer threads, keep fluids out of dead space before connecting to feeding tube.

For best results, follow these instructions to clean tubes at least once a day or whenever material is visible.

Tube Cleaning Supplies & Terms				
 Cup of clean water	 Syringe	 Gauze	 Brush* or ENFit specific cleaning tool	 ENFit Feeding Tube
Note: Use a disposable brush or follow manufacturer's instructions if using ENFit specific cleaning brush.				
 1 Wash hands with soap and water. Rinse brush with tap water.		 2 Fill syringe with water.		
 3 Plug center hole of feeding tube port with brush bristles. Forcefully flush moat with water.		 4 Rotate brush in bottom of moat.		 5 Rinse cap with clean tap water.
 6 Insert bristles into feeding tube cap and rotate brush in cap to clean.			 7 Wipe feeding tube port and cap with gauze. Clean supplies and allow to air dry.	

Repeat steps 3 through 6 until cap and tube are thoroughly clean.

* A manual toothbrush is regulated as a medical device intended to remove debris from the teeth in some jurisdictions. Consult your licensed healthcare provider or Risk Manager regarding recommended use for cleaning feeding tube ports. Dispose of single use devices as instructed. Cleaning procedures courtesy of Children's Mercy Kansas City.
© GEDSA 2018. ENFit is a registered trademark of GEDSA.

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